


DECRON™

QuickCast Leg Splint

- Vital first-aid to immobilise and support leg injuries at any scene.
- Activates with water, mouldable for perfect conformation, sets rock hard.

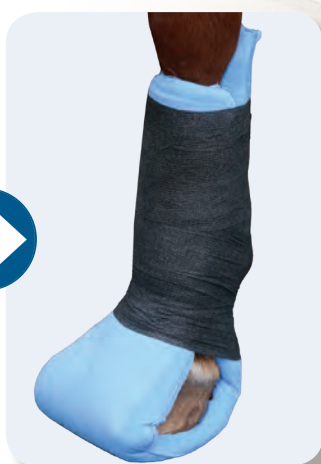

Secure, customised stabilisation.



**IMMOBILISATION
AT THE SCENE OR
POST-SURGERY**

Ideal for first response or post-operative support.



**ACTIVATES
QUICKLY WITH
WATER**

Easy to mould for custom fit in any location.
Sets fast.



**FORMS A STRONG,
SUPPORTIVE
CAST**

Rock hard outer shell stabilises.
Inner beads conform while setting.
Soft cover prevents abrasion.



**EASY TO REMOVE
AND REAPPLY FOR
WOUND TREATMENT**

Allows quick, safe access without compromising care.



CORRECT CROOKED FOALS WITH NEW FOAL SIZES

DECRON™
HORSE CARE

DECRON HORSE CARE PTY LTD
Product enquiries: Simon 0408 640 101
Sales: support@decron.com.au

www.decron.com.au

SCAN
FOR MORE INFO



DECRON™

QuickCast Leg Splint

EXPERT TIPS

Size can be tailored.

- For horses shorter than your splint, simply fold splint at one or both ends and bandage.
- Fold immediately upon application before outer shell begins to harden.

For instant application:

1. Pre-cast on a sound, similar sized, horse.
 2. Transfer to target horse and wrap.
- Minor mismatch in donor horse size/shape can be negated by QuickCast's inner beads which provide adaptable conformation.
 - If donor horse is significantly undersized, thickly pre-wrap donor prior to casting.
 - Save more time: soft outer fabric prevents abrasion and pressure sores, no need to pre-wrap or pad.

No water? No problem.

QuickCast will set slowly when exposed to humidity. Water is strongly recommended for faster setting.



CURRENTLY IN USE AND TRIALLING

Scone Equine
Goulburn Valley Equine Hospital
Wollondilly Equine
Rood & Riddle, Lexington Kentucky, USA
Crown Equine Veterinary Services
Martillia Horse, Italy
Many more

Watch application video from
Dr Denis Verwilghen of the
Goulburn Valley Equine
Hospital



LEARN FROM THE QUICKCAST PROFESSIONAL DEVELOPMENT NETWORK

Vets, nurses, farriers, trainers with hands-on experience are available to help, and learn from each other.

- Specific use cases and indications
- Techniques from proven practice
- Sizing experience
- Unique advantages over Kimzey splint, Robert Jones, fibreglass, and PVC

Register your interest at
[decron.com.au/
QuickCastNetwork](http://decron.com.au/QuickCastNetwork)



AVAILABLE SIZES

QCLS140	Decron QuickCast Leg Splint - Large	15 x 140cm
QCLS120	Decron QuickCast Leg Splint - Medium	15 x 120cm
QCLS100	Decron QuickCast Leg Splint - Small	15 x 100cm
QCLS100N	Decron QuickCast Leg Splint - Foal Small	12.5 x 100cm
Further sizes coming and also available on request		



DECRON™
HORSE CARE